Apicold[®] mint

spray do nosa

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Apicold[®] mint

nasal spray

ISOTONIC SOLUTION FOR NASAL HUMIDIFICATION AND
CLEANSING

Medical device

Composition: AQUA, SODIUM CHLORIDE, CALCIUM CHLORIDE, POTASSIUM CHLORIDE, EUCALYPTUS GLOBULUS OIL, MENTHOL, DISODIUM PHOSPHATE, BENZALKONIUM CHLORIDE

Intended use

Isotonic solution for nasal humidification and cleansing. Apicold[®] mint nasal spray normalizes the function of nasal mucous membrane on a natural way. Finely dispersed isotonic solution, allows action on the entire surface of the nasal mucous membrane. Its composition helps to establish the natural pH and nasal mucosa hydration, as well as the dissolution of dried mucous and accumulated dirt for easier breathing. The addition of menthol and eucalyptus oils contributes to a pleasant feeling of freshness after use.

It is recommended

- for daily hygiene of mucous membrane of children and adults
- while staying in air-conditioned rooms
- while exposed to dust and harmful atmospheric effects
- while staying in dry and warm rooms (central heating)
- for symptoms of allergic rhinitis and colds

Instructions for use

Remove the protective cap. Insert the tip of the nasal spray into one nostril, press the pump and breathe gently. For the hydration of the nasal mucous membrane, one spray per nostril several times a day is sufficient. To treat nasal congestion, spray Apicold[®] mint once or twice into both nostrils and repeat several times a day.

Caution

Keep out of reach of children. Not recommended for children under the age of 4. Due to hygienic reasons, it is recommended that one bottle is used for one person only. In case of hypersensitivity to certain ingredients, the use of Apicold[®] mint spray is not recommended. It is recommended to use the product within 30 days of opening. For external use only.

Safekeeping

Store in dry and dark place at room temperature.

Net quantity

30 ml

Lot number and expiry date are printed on the bottom of the packaging.

Explanation of the symbols appearing on the packaging

	Consult instruction for use
	Expiry date
	Date of manufacture
	Warning
	Manufacturer
	Lot number
	Reference number
	Dispose carefully

APIPHARMA d.o.o.
Jeronima Kavanjina 26,
10090 Zagreb,
Croatia, EU

Produced for:
Alpen Pharma AG,
Casinoplatz 2, 3011
Bern, Switzerland
Distributor:
Alpen Pharma
Polska Sp. z o.o.,
Al. Jerozolimskie 151/1,
02-326 Warszawa

EN_PL-10MD